

Shipping Instructions

Shipping

Attach the enclosed pre-paid Royal Mail address / postage label to the box and place the box in a post box or take to a Post Office for dispatch.

Lab 4 More Urine Instructions



These instructions are for use from within the UK only.

Please keep the box provided to return your sample in.

| All Others |
|---------------------------------------|
| 1 x Urine Collection Tube |
| Clear Zip Lock Back |
| 1 x Royal Mail pre-paid returns label |
| 1 x Requisition form |
| 1 x security seal |
| 1 x Return Shipping Label |
| |

Patient Information

When?:

Urine should be collected as a first morning void (sample).

Advanced Preparation

- Only undertake this test Mon to Wednesday.
- This sample must be shipped by Royal Mail the same day.

Collection

Minimum sample required is 10ml.

Labelling

- Please ensure you use the attached patient identifier label on the sample tube and add the date of collection.
- Make certain your requisition form is completed and that the name matches exactly the tube on your urine sample.

Packing

- Place all completed and labelled tubes into hazard bag and seal.
- Place bag into return shipping box (same box as kit arrived in) with completed paperwork.
- Seal Box with the security seal provided.
- This specimen is ready for shipping and must be dispatched the same day as collection, to arrive at Regenerus Laboratories Monday to Wednesday only.

For assistance call 0203 750 0870