

Shipping and posting information:

Because of overseas shipping requirements, we do not receive samples on Weekends or Bank Holidays Please ensure your samples are received by us no later than Friday Middy.

Due to the nature of this specimen and its volume, you are not allowed to send this sample through the Royal Mail postal system. Please use the UPS service offered or another of your choice.

For health and safety reasons, you must use all the packaging enclosed and seal each bag correctly before returning your samples.

Regenerus Laboratories reserves the right to dispose of any sample received which does not meet this criteria.

Returning your samples in the UK via UPS

Enclosed you will find a return shipping label for our UPS courier service. Should you choose to use this service then you will be charged accordingly, details of which are separately enclosed.

Within the UK, UPS offer 3000 convenient drop off locations, this service means no waiting for couriers to arrive, we suggest you visit the UPS website at www.ups.com/dropoff. Alternatively the UPS home /office collection service normally offer a 4 hour collection window. Please see the enclosed shipping information sheet.

Track the status of your shipment, copy the tracking number on your shipping label and track its progress using the UPS website.

Regenerus Laboratories do not keep records of your tracking number.

Hepatic Detox Profile**Before You Start:**

Please read all of the directions, and familiarize yourself with the collection procedures. A first morning void urine collection is recommended for the Hepatic Detox Profile.

The test requires no special diet but if you are taking sulfur containing medications or dietary supplements such as DMSA, DMPS, MSM, etc., please finish the course of medication, and then wait 4 days before starting this collection. A positive (or elevated) level of D-Glucaric Acid may result if you are using medications such as barbiturates, tranquilizers digoxins, quinidine, or alcohol. Please refrain from taking non-essential medications, dietary supplements, and alcohol for 72 hours prior to and during the specimen collection, unless otherwise instructed by your physician. *Never discontinue prescription medications without first consulting your physician.*

Female patients should not collect urine during a menstrual period.

Verify Kit Contents:

- 1 Specimen collection cup
- 1 Specimen vial
- 1 Zip-lock bag with absorbent material
- 1 Test requisition form
- 1 Return Box, Blue Shipping Bag, UPS Shipping Label

If you are missing kit components, please call Regenerus Laboratories Customer Service department for assistance.

Regenerus Laboratories Limited

Tel: 0333 9000 979

email: info@regeneruslabs.com

Aero 14, Redhill Aerodrome, Kings Mill Lane, Redhill, RH1 5YP
Company Registered in England and Wales No. 08077110

Urine Collection Instructions:

1) When you arise collect your first morning urination into the specimen collection cup. Pour the urine into the specimen vial. Fill the vial up to the top of the label and then tighten the screw cap securely .

2) In the collection information section of the requisition form (located in the upper right hand corner) fill in the date you finished the collection. Write the patient's name, the date of collection, and patient's date of birth on the specimen vial. The test cannot be performed without the patient information on the vial. Place the vial into the zip-lock bag, then place the bag in the return box.

3) Fill out the test requisition form completely and sign it. The test cannot be performed without a properly filled out requisition. Place the completed requisition form in the return box. The specimen is now ready for shipment.

To obtain the best results, Doctor's Data recommends shipping specimens as soon as possible. If you cannot arrange shipping on the same day you finish the collection, such as over a weekend or holiday, refrigerate the specimen until you are ready to ship .

Consult your physician if you have any questions during the test collection process.